**SOLUBRUX®**

**SOLUBRUX®**

**SoluBrux®** is the first dynamic mouldable bite splint for immediate use. Individual adjustment should be made by a dentist, but can also be made by patients themselves under compliance with the following instructions. Correctly moulded, it offers a high degree of comfort and protection against the consequences of bruxism (teeth grinding and pressing).

**Package contents:**

- Figure A: 1 **SoluBrux®**, anatomically formed bite splint
- Figure B: 1 storage box
- Figure C: 1 holding fork
- Figure D: 1 bite liner

All utilised materials are non-toxic, anallergic and approved for use in the "oral" cavity. **SoluBrux®** is patented and marked with the symbol **CE** as a Class I medicinal device.

**Instructions for use:**





- Individual moulding is generally carried out on the upper jaw teeth.
- 1 - Connect the holding fork (Figure C) to the bite liner (Figure D). Caution, make sure the holding fork has the marking facing upward!
  - 2 - Insert the combination bite liner/holding fork all the way into the provided apertures on the front side of **SoluBrux®** (see graphic 1).
  - 3 - Now insert **SoluBrux®** with bite liner in the mouth before heating up. This is important for two reasons: 1. Should you feel an urge to gag, the splint can be shortened (nail scissors, scalpel). 2. For perfect moulding (see Point 6), it is important to press the tongue to the gums and remove the saliva through suction, to generate a vacuum. This is why this should be practiced beforehand.
  - 4 - Pour enough boiling water to cover the **SoluBrux®** into a container and with the help of the holding fork, immerse **SoluBrux®** and bite liner for 12-15 seconds. Please use a stopwatch or a watch with a second hand for control! (graphic 2)
  - 5 - Remove **SoluBrux®** from the container and shake off any excess water. Before inserting in the mouth, make sure that the splint is entirely open. If necessary, "bend up" and open a bit with the fingers in order to prevent the edges from being pressed down during insertion and biting, which would make the splint useless. Please note that the instruction in the above point must be followed quickly in order to prevent the material from cooling down and becoming un-mouldable.
  - 6 - Insert **SoluBrux®** carefully into mouth, close mouth, and clench teeth firmly. Generate a vacuum (Please refer to point 3 (2) above) so that the warm, soft material adjusts to the contours of your teeth.
  - 7 - Hold the "vacuum" as long as possible. After 1-2 minutes, remove **SoluBrux®** from mouth and place in cold water for several minutes. Remove the holding fork and the bite liner and store in a safe place until needed again should an re-moulding be necessary.

**Important references**

- In the event of malfunctions of the dental or dental holding apparatus, extreme malposition of teeth, wearing of removable dentures or dental braces, ask your dentist before using **SoluBrux®**.
- If the symptoms associated with bruxism do not improve or even get worse within 2-3 weeks, despite regularly wearing **SoluBrux®**, consult your dentist.
- Brush **SoluBrux®** with either toothpaste or soap in cold water. Fast-acting cleansing tablets or ultrasonic bath can also be used.
- Never clean **SoluBrux®** with hot water!
- Do not store **SoluBrux®** in the proximity of heat sources.
- **SoluBrux®** is not suitable for milk teeth.

More information: [www.solubrux.com](http://www.solubrux.com)

**SOLUBRUX®** is a registered brand name

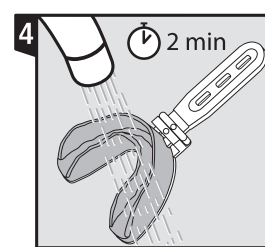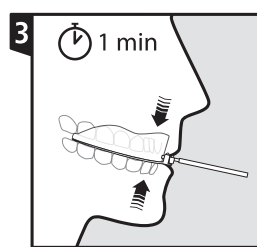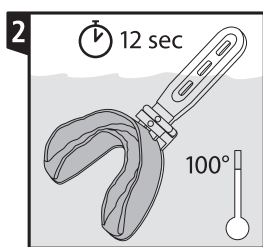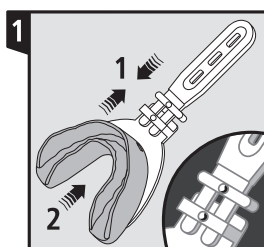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